



## **NUFLOW ORAL FLUID EXTRACTION DEVICE**

### **INSTRUCTIONS FOR USE**

#### Indications for Use:

The NuFlow Oral Fluid Extraction device is a single use device designed to provide continuous fluid removal under vacuum during oral surgery.

#### Contraindications:

- Do not use it if the patient is known or suspected of being allergic to materials contained in the device.

#### Warnings:

- Verify patients have been cleared by an anesthesiologist and physician to proceed with the use of NuFlow product.
- Do not reuse. Intended for single patient use only.
- Do not use it if product or package is damaged.
- Sterilized by Ethylene Oxide. Do not resterilize.

#### Precautions:

- Caution: Federal (USA) law restricts this device to sale by or on the order of a physician. Use of this device is only to be completed by qualified health professionals.

### **Directions for Use:**

1. Visually inspect product and packaging before use.
2. Open the NuFlow device pouch in a sterile surgical field.
3. Handle NuFlow device with sterile gloves.
4. Soak the NuFlow sponge in sterile saline to activate or until the sponge is soft to touch with a glove.
5. While the sponge is submerged in the saline, connect the receiving port of NuFlow device into the facility's vacuum suction apparatus. Turn on the vacuum suction unit. Verify the NuFlow device is functioning correctly by visualizing liquid being removed.
6. Once the patient is properly prepared for surgery by the clinician, place the NuFlow sponge in the oropharynx, positioning it behind the soft palette at the base of the tongue. **PROPER POSITIONING IS IMPORTANT.**
7. Secure the NuFlow tubing on the sides of the mouth by placing it between the retractor and the cheek. This will ensure the NuFlow device does not obstruct the surgical field during surgery.
8. Verify the vacuum is functioning properly and proceed with surgery. Examine the NuFlow device periodically. Maintain airway as required in all procedures.
9. After surgery is complete, turn off the vacuum and remove the device by grabbing the tubing and gently pulling NuFlow device out. Disconnect the vacuum and throw away disposable device in proper biohazard containers.



Sterility:

This device has been sterilized and is for single use only and is not to be reused. As long as the packaging remains sealed and uncompromised, the contents within the package are sterile. NuFlow Products will not be responsible for any products that are resterilized, nor accept any exchange or credit for product that has been opened but not used.

Disposal After Use:

After use, medical products pose a potential biohazard. For this reason, the product should be handled and disposed of in accordance with recognized medical procedures and in compliance with applicable local, state and federal laws and regulations

Warranty:

NuFlow Products warrants that this medical device is free from defects. The above warranty are in lieu of all other warranties, either expressed or implied, including any warranty of merchantability or fitness for a particular purpose. Suitability for use of the medical device for the procedure should be determined by the user. NuFlow Products shall not be liable for incidental or consequential damages of any kind.



NuFlow Products, Inc.  
1519 Madrid Drive  
Vista, CA 92081 USA  
[www.nuflowinc.com](http://www.nuflowinc.com)  
619-832-9571  
Admin@nuflowinc.com

Symbols Glossary:



Manufacturer




Do not reuse



Do not use if package is damaged and consult instructions for use



Use by date

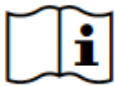

Consult instructions for use



Do not re sterilize



Batch code



Catalogue number



For prescription use only